**BAC** S.R.L.

SIRIO

Sirio  
BAC | italy  
medical

MEDICAL TECHNOLOGY

MADE IN ITALY

# SIRIO

## MEDICAL TECHNOLOGY

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***SIRIO is a medical device certified in class IIb according to the directive 93/42/EEC. The device has been designed and developed for applications both in aesthetic medicine and physiotherapy***

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*SIRIO features an elegant ergonomic design. The most advanced technologies employed allow its compact dimensions and lightweight and make it very easy to handle and move around.*

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*SIRIO is 100% made in Italy. It is engineered, designed, manufactured and assembled in Italy to guarantee the highest quality standards, safety and attention to details.*

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*When a hand probe is connected to the device, SIRIO automatically starts a self-recognition procedure in order to work in total safety. Besides the supplied hand probes, other styles of hand probes are available for different treatments in research and testing phase.*

SIRIO can be supplied with:

- 2 hand probes (1 flat and 1 concave hand probe) for applications in aesthetic medicine,
- 2 hand probes (1 flat and 1 concave hand probe) for applications in physiotherapy,
- 4 hand probes (2 flat and 2 concave hand probes) for applications in aesthetic medicine and in physiotherapy.

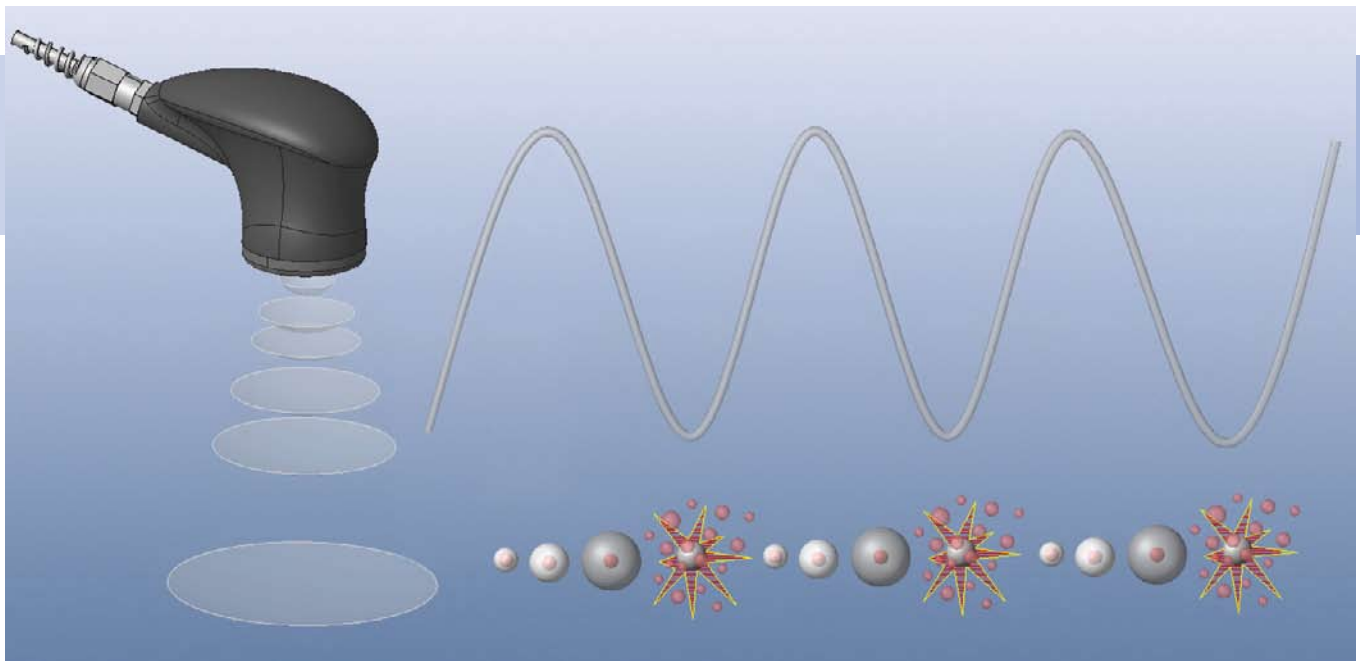
Before and during the treatment our software automatically performs the following tasks:

- recognition and qualification of hand probes
- data storage
- broad spectrum verification of efficiency.

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*The device is supplied with a class II external power supply, which carries IEC G0601-1 medical approval. Different adapter plugs are available for use in Europe, US, UK, Australia and Japan.*

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### HOW SIRIO WORKS

*SIRIO medical device is a low frequency (38 KHz +/- 2KHz) ultrasound generator; it is run by a micro-processor which can produce acoustic waves in continuous or pulsed mode and simultaneously can monitor its full revolution cycle. It adjusts and lowers the power output in case of excessive adsorption of the hand probe.*

*Sirio is supplied with hand probes for non-invasive treatments, which can be flat or concave.*

*Sirio effectiveness depends on its ability to take advantage in a more penetrating way of the different effects typical of ultrasounds (mechanical, vibrational, thermal, cavitation) inside the tissues because the low frequency emission let a deeper action.*

*There are two main fields of application:*

*Physiotherapy: affection of the musculoskeletal system, as, for instance, joint pathologies (shoulder, knee and ankle areas), muscle injuries or contractures, meniscus pains, pains resulting from ankle sprain, tendon pains, etc.*

*Aesthetic Medicine: aesthetic problems related to cellulite, localized adiposity and lymphatic stagnation (oedema).*

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SIRIO must not be used in certain indicated areas. It is mandatory to follow our treatment protocols, which have been extensively studied and tested. These protocols guarantee the best results in total safety.

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## TECHNICAL FEATURES

Type of device: 38 kHz +/-2 Ultrasound device  
Conformity and standards: 93/42/CEE, 2007/47/CE  
Supply: 24 VDC – 2,9 A  
Supply voltage: 100/240 V - 50/60 Hz  
Absorbed power: 60 W max.  
Display: LED backlight touch screen  
Size: L 315 x D 220 x H 220  
Weight: 4,7 Kg approx.  
Hand probe power: Max. 3W/cm<sup>2</sup>  
Transducer hand probe size: Ø 1,97 inch / 3,08 inch<sup>2</sup> approx.  
Hand probe weight: 700 g approx  
Sale packaging: Cardboard box  
IEC 60601-1 classification: Class II, Application parts BF Type, IP XO

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Password protection  
Energy efficiency  
Continuous data exchange between HW and SW  
System automatically stops in case of inefficiency  
Power output is constantly monitored and optimized  
Treatment time and performance are optimized  
Protection against electromagnetic emissions  
Constant monitoring and adjustment of resonant frequency  
Full traceability of the device and its individual components

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*Several precautions were taken during hardware and software development to fulfill the strictest standards in the industry. Furthermore during production and assembly several process checks are carried out to ensure conformity to the regulation of reference. SIRIO is a medical device certified in class IIb according to the directive 93/42/EEC.*

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To easily transport SIRIO around, CUOIERIA FIORENTINA produces on purpose a specific trolley case. It is made of vegetable tan leather which is sewn with the same methods of ancient craftsman of Florence.

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